


REAL ELECTRONICS


REAL 9000 PLUS

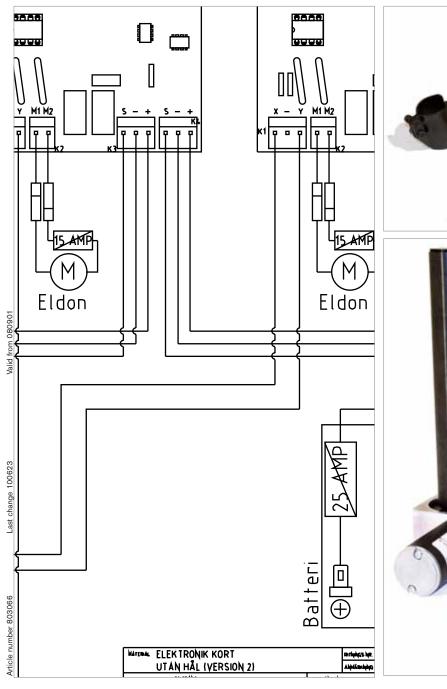









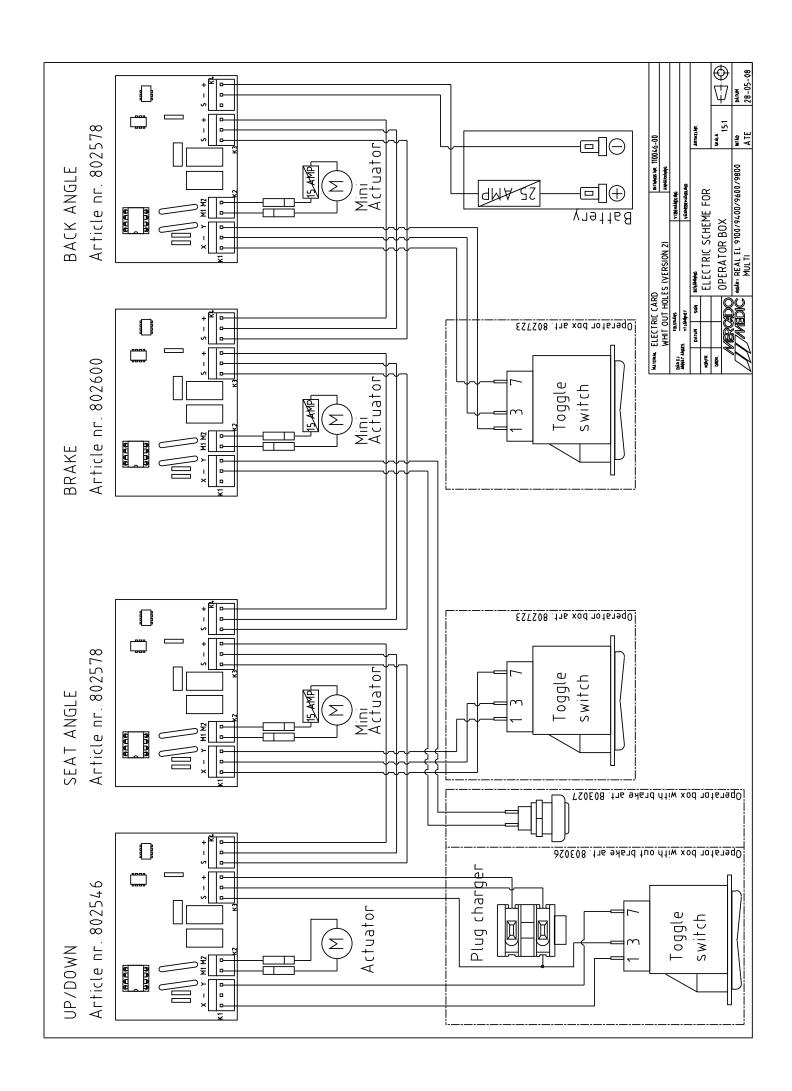


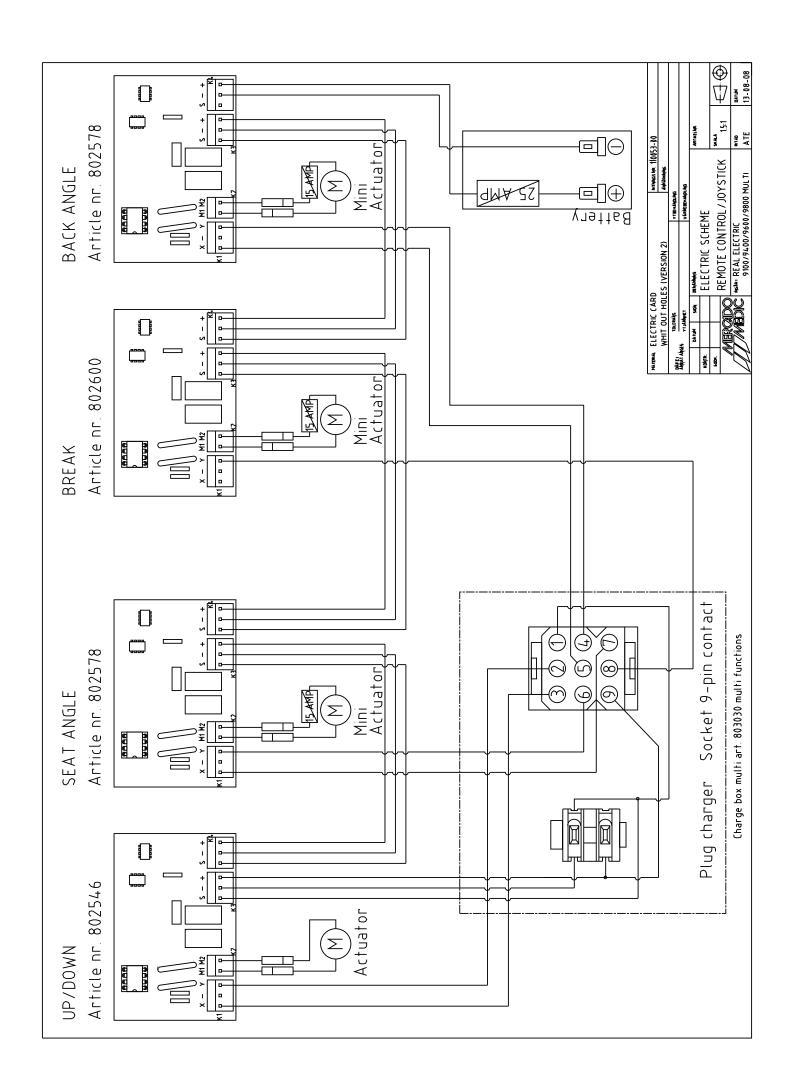



MOUNTING OF RELAY CARD KIT 1 Mount the bolster plate with the screws on to the shell of the Liftomat with the arrow pointing upwards towardsthe pipe of the Liftomat. On the Electric kit the bolster plate shall be mounted on the back side of the perpendicular plate angle, with the arrow pointing up. 2 Attach the lid over the box with two plastic straps. 3 Shove the box down in the slots of the bolster plate, until the catch fastens.

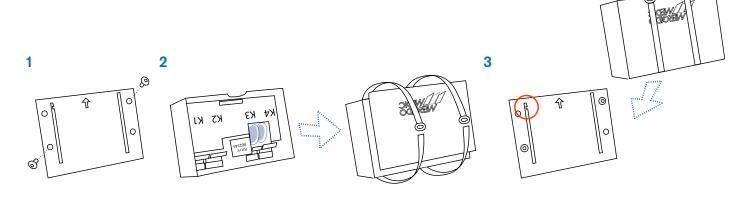

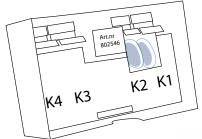

IF YOU WANT TO BUILD IT YOURSELF, DO LIKE THIS: ON THE CIRCUIT BOARD YOU CAN FIND THE MOLEXCONTACTS K1-K4.



WE RECOMMEND YOU TO ORDER THIS KIT WITH ART.NO 802998. IT INCLUDES EVERYTHING.





CHARGE BOX UP/DOWN STEP 1

To connect the Charge box up/down art.no 803029. Engage the contact with TWO leaders to the contact on the circuit board marked K1 and the contact with THREE leaders to the contact marked K3.





CHARGE BOX MULTI STEP 1

To connect the Charge box multi art.no 803030.

Use the cord with TWO contacts. Engage the contact with TWO leaders to the contact on the circuit board marked K1 and the contact with THREE leaders to the contact marked K3.





MANEUVER BOX STEP 1

To connect the Maneuver box art.no 803034. Engage the contact with TWO leaders to the contact on the circuit board marked K1 and the contact with THREE leaders to the contact marked K3.





To connect the Cabling for Motor art.no 802681. Engage the contact to the contact on the circuit board marked K2. The other end is to be connected to the engine.







► CHARGE BOX ■ MANEUVER BOX STEP 3

MERCADOS NEW ELCTRIC KIT & ELECTRONICS FOR SEAT ANGLE

Since the PLUS electronics are minuscontrolled they're not backwards compatible. Electronics Version 1 & Version 2 can be combined with the electronics of PLUS.

The image indicates the appliances 803000 and 802730 mounted on to a seatframe that is adjustable in angle.

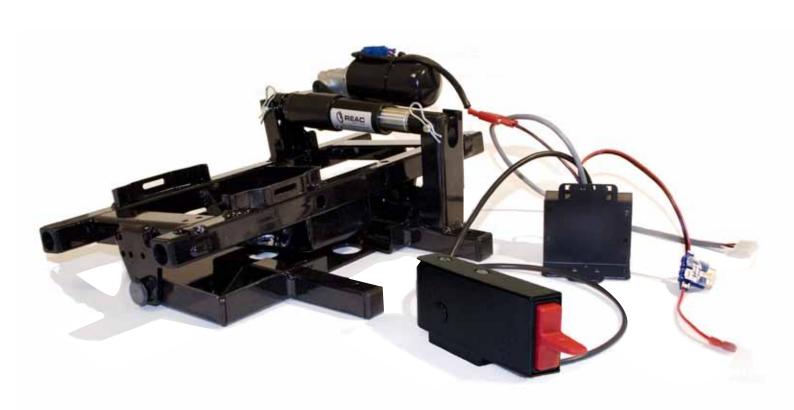

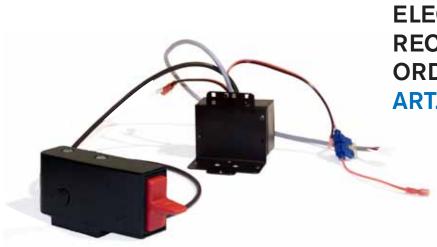

TO COMPLETE A CHAIR WITH ELECTRIC SEAT ANGLE, WE RECOMMEND YOU TO ORDER THE TWO KITS -

ART.NO 803000 & 802730





CHARGE BOX MULTI STEP 1

To connect the Charge box multi art.no 803030. Engage the contact marked S to the contact on the circuit board marked K1.





MANEUVER BOX STEP 1

To connect the Maneuver box art.no 803035. Engage the contact to the contact on the circuit board marked K1.





CHARGE BOX MANEUVER BOX STEP 2

To connect the Cabling for Motor art.no 802681. Engage the contact to the contact on the circuit board marked K2. The other end is to be connected to the engine.





CHARGE BOX MANEUVERBOX STEP 3

To connect the Cabling for Electricity supply art.no 802718. Engage the contact to the contact on the circuit board marked K3. NOTE! The cable comes from the circuit board for Up & Down or prior circuit board.






CHARGE BOX MANEUVER BOX STEP 4

NEW ELCTRIC KIT & ELECTRONICS FOR BACKREST ANGLE

Since the PLUS electronics are minuscontrolled they're not backwards compatible. Electronics Version 1 & Version 2 can be combined with the electronics of PLUS.

The image indicates the appliances 802999 & 802731.

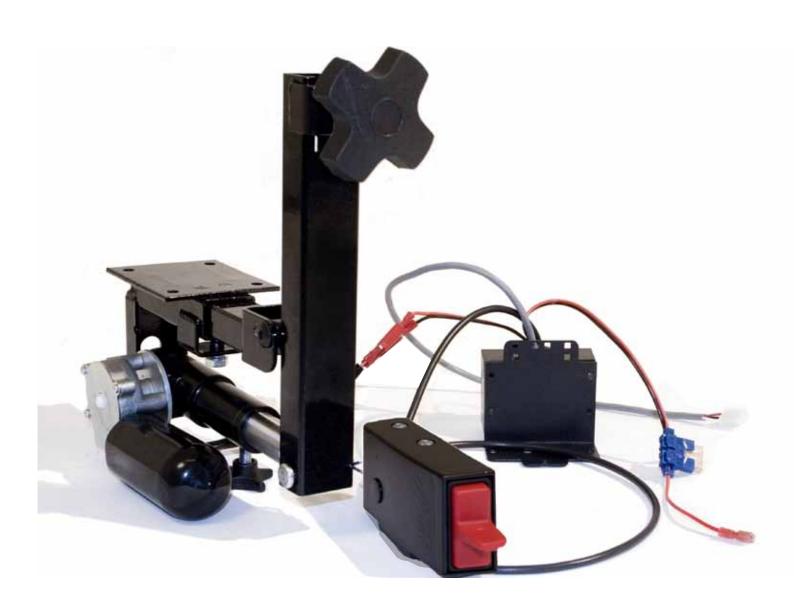





CHARGE BOX MULTI STEP 1

To connect the Charge box multi art.no 803030. Engage the contact marked R to the contact on the circuit board marked K1.

MANEUVER BOX STEP 1

To connect the Maneuver box art.no 803035. Engage the contact to the contact on the circuit board marked K1.

CHARGE BOX MANEUVER BOX STEP 2

To connect the Cabling for Motor art.no 802681. Engage the contact to the contact on the circuit board marked K2. The other end is to be connected to the engine.

CHARGE BOX MANEUVER BOX STEP 3

To connect the Cabling for Electricity supply art.no 802718. Engage the contact to the contact on the circuit board marked K3. NOTE! The cable comes from the circuit board for Up & Down or prior circuit board.

CHARGE BOX MANEUVER BOX STEP 4


THE NEW ELECTRIC BRAKE FROM MERCADO

Since the PLUS electronics are minuscontrolled they're not backwards compatible. Electronics Version 1 & Version 2 can be combined with the electronics of PLUS.

Assembly instructions can be ordered separately from Mercado or be downloaded from www.mercado.se

The image indicates the appliances 803001 & 803037.



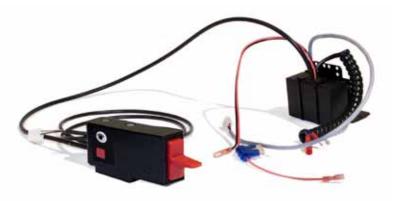

TO COMPLETE A CHAIR WITH AN ELECTRIC BRAKE, WE RECOMMEND YOU TO ORDER THESE KITS - ART.NO 803001 & 803037



CHARGE BOX MULTI STEP 1

To connect the Charge box multi art.no 803030. Engage the contact marked B to the contact on the circuit board marked K1.

MANEUVER BOX STEP 1

To connect the Maneuver box with brake button art.no 803036 (replaces the maneuver box up/down 803034). Connect the cable with ONE contact to the contact on the circuit board marked K1. Connect the cable with TWO contacts to the relay card for up/down.

◆ CHARGE BOX ◆ MANEUVER BOX STEP 2

To connect the Cabling for Motor art.no 802719. Engage the contact to the contact on the circuit board marked K2. The other end is to be connected to the engine.

CHARGE BOX MANEUVER BOX STEP 3

To connect the Cabling for Electricity supply art.no 802718. Engage the contact to the contact on the circuit board marked K3. NOTE! The cable comes from the circuit board for Up & Down or prior circuit board.

CHARGE BOX MANEUVER BOX STEP 4

ARTICLE LISTING OVER THE KITS FOR MANEUVER BOX

802998 Relay card kit f up/down incl. Maneuver box 9000 PLUS

Art.no	Term	Amount
803034	Maneuver box Plus f up/down+charger cpl with cabling f Relay card	1 PCS
802681	Cabling f motor 9100 f Relay card	1 PCS
802648	Cabling f battery cpl 9100 f Relay card	1 PCS
803018	Cabling lead-through 10x14x1.5	1 PCS
802696	Maneuver button f Relay card	1 PCS
800774	Plastic plug Man box Electric chair 50x30	1 PCS
802546	Relay card current limitation Up-Down REAL9000	1 PCS
802832	Relay card box f Relay card	1 PCS
802835	Protective cover t Relay card box f Relay card	1 PCS
802833	Cover t Relay card box f Relay card	1 PCS
802834	Attachment t Relay card box f Relay card	1 PCS
800438	K6S M4x6 OBH Socket head cap screw M4x6 oval head screw	2 PCS
800023	Bundle strap 203x3.6mm black	2 PCS

802999 Relay card kit f backrest angle incl. Maneuver box 9000 PLUS

Art.no	Term	Amount
803035	Maneuver box Plus f Seat /Backrest angle with cabling f Relay card	1 PCS
802681	Cabling f motor 9100 f Relay card	1 PCS
802718	Cabling f electricity supply f Relay card L=500mm	1 PCS
803019	Cable lead-through 11x14x1.5	1 PCS
802696	Maneuver button f Relay card	1 PCS
800774	Plastic plug Man box Electric chair 50x30	1 PCS
801070	Plastic stopper height adjust. arm -support	1 PCS
802578	Relay card current limitation Seat/Backangle 16 amp	1 PCS
802832	Relay card box f Relay card	1 PCS
802835	Protective cover t Relay card box f Relay card	1 PCS
802833	Cover t Relay card box f Relay card	1 PCS
802834	Attachment t Relay card box f Relay card	1 PCS
800438	K6S M4x6 OBH Socket head cap screw M4x6 oval head screw	2 PCS
802714	Bolster plate f Relay card	1 PCS
802862	MRT-TT FZB M4x6 (screw for charge box)	2 PCS
800023	Bundle strap 203x3.6mm black	4 PCS
800057	Bundle strap 71x1.8mm black	3 PCS

802731 Electric kit f backrest angle f relay card Real 9000

Art.no	Term	Amount
802644	Electric kit REAL f Backrest angle f Real 9000	1 PCS
801354	Cylindrical bolt FZB	2 PCS
801725	Clasp pin 2mm FZB Type B f Cylindrical bolt	2 PCS
801421	Electrical back attachment, adjustable Rear side	1 PCS
801422	Electrical back attachment, adjustable Front side	1 PCS
802744	Electrical back, back pipe lower f Electric kit	1 PCS
800837	Back tilting attachment Medic Cla/ErgoM	1 PCS
800369	M6S M8x16 8.8 FZB Screw M8x16 FZB	1 PCS
801212	Furniture Nut M6 Rough skull	2 PCS
800507	MPSH M6x38 FZB Pin screw M6x38	1 PCS
801566	MF6S M6x16 OBH 10.9 Socket sink head M6x16	2 PCS
800096	MC6S M6x16 FZB 8.8 Socket head cap screw M6x16 FZB	4 PCS
800462	BRB 6.4x12x1.5 black Plate 6.4x12x1.5 FZB	4 PCS

801265	needed; complete the electric kit with one of following back pillars Back pillar Std. Cla/ErgoM Low back	1 PCS
800841	Back pillar Std. Cla/ErgoM High back	1 PCS
8030	000 Relay card kit f seat angle incl. Maneu	iver box 9000 PLUS
803035	Maneuver box Plus f Seat /Backangle with cabling f Relay card	1 PCS
802681	Cabling f motor 9100 f Relay card	1 PCS
802718	Cabling f electricity supply f Relay card L=500mm	1 PCS
803019	Cable lead-through 11x14x1.5	1 PCS
802696	Maneuver button f Relay card	1 PCS
800774	Plastic plug Man box Electric chair 50x30	1 PCS
801070	Plastic stopper height adjust. arm -support	1 PCS
802578	Relay card current limitation Seat/Backangle 16 amp Relay card box f Relay card	1 PCS 1 PCS
802832 802835	Protective cover t Relay card box f Relay card	1 PCS
802833	Cover t Relay card box f Relay card	1 PCS
802834	Attachment t Relay card box f Relay card	1 PCS
800438	K6S M4x6 OBH Socket head cap screw M4x6 oval head screw	2 PCS
802714	Bolster plate f Relay card	1 PCS
802862	MRT-TT FZB M4x6 (screw for charge box)	2 PCS
800023	Bundle strap 203x3.6mm black	4 PCS
800057	Bundle strap 71x1.8mm black	3 PCS
802643 801354	Electric kit REAL f Seat angle f Real 9000 Cylindrical bolt FZB	1 PCS 2 PCS
	5	
801725	Clasp pin 2mm FZB Type B f Cylindrical bolt	2 PCS
	001 Delay card kit f brake col incl. maneus	er hay 9000 DILIS
Art.no	OO1 Relay card kit f brake cpl incl. maneuv Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card	Amount
Art.no 803036	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card	Amount 1 PCS
Art.no 803036 802719	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable)	Amount 1 PCS 1 PCS
Art.no 803036 802719 802718	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card	Amount 1 PCS
Art.no 803036 802719 802718 802642	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm	Amount 1 PCS 1 PCS 1 PCS
Art.no 803036 802719 802718 802642 803019	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard	Amount 1 PCS 1 PCS 1 PCS 1 PCS
Art.no 803036 802719 802718 802642 803019 802696	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5	Amount 1 PCS 1 PCS 1 PCS 1 PCS 1 PCS 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30	Amount 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp	Amount 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card	Amount 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card	Amount 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802833	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card	Amount 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802834	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card	Amount 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802834 802834	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw	Amount 1 PCS 2 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802834 800438 802714	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card	Amount 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802834 800438 802714 800436	Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt	Amount 1 PCS 2 PCS 2 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802834 800436 800436 800023	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt Bundle strap 203x3.6mm black	Amount 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802835 802834 800438 80023 800057	Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt Bundle strap 203x3.6mm black Bundle strap 71x1.8mm black	Amount 1 PCS 2 PCS 1 PCS 2 PCS 4 PCS 3 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 8002600 802832 802835 802834 800438 802714 800436 800023 800057	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt Bundle strap 203x3.6mm black	Amount 1 PCS 2 PCS 1 PCS 2 PCS 4 PCS 3 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802835 802834 800438 800714 800436 800023 800057 B03 (Art.no	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt Bundle strap 203x3.6mm black Bundle strap 71x1.8mm black D37 Electric kit f electric brake PLUS-base	Amount 1 PCS 2 PCS 4 PCS 3 PCS 2 PCS 4 PCS 3 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802835 802834 800438 802714 800436 800023 800057 BO3C Art.no 802645	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt Bundle strap 203x3.6mm black Bundle strap 71x1.8mm black D37 Electric kit f electric brake PLUS-base Term	Amount 1 PCS 2 PCS 1 PCS 2 PCS 4 PCS 3 PCS 4 PCS 3 PCS 4 Mount
Art.no 803036 802719 802718 802642 803019 802696 800158 8002600 802832 802835 802834 800438 802714 800436 800023 800057 8036 Art.no 802645 800180	Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt Bundle strap 203x3.6mm black Bundle strap 71x1.8mm black 37 Electric kit f electric brake PLUS-base Term Electric kit REAL f brake f Real 9000	Amount 1 PCS 2 PCS 4 PCS 3 PCS 2 Manual 4 Manual 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 8002632 802835 802835 802834 800438 802714 800436 800023 800057 B03(Art.no 802645 800180 800181	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt Bundle strap 203x3.6mm black Bundle strap 71x1.8mm black D37 Electric kit f electric brake PLUS-base Term Electric kit REAL f brake f Real 9000 Round pin socket Red 1504 HO Red 0.75-1.65 mm2	Amount 1 PCS 2 PCS 1 PCS 2 PCS 4 PCS 3 PCS 4 PCS 3 PCS 4 PCS 3 PCS 4 PCS 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802833 802834 800438 800023 800057 BO3(Art.no 802645 800181 802705	Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt Bundle strap 203x3.6mm black Bundle strap 71x1.8mm black D37 Electric kit f electric brake PLUS-base Term Electric kit REAL f brake f Real 9000 Round pin socket Red 1504 HO Red 0.75-1.65 mm2 Round pin Red 1504 HA 0.75-1.65 mm2	Amount 1 PCS 2 PCS 4 PCS 3 PCS 4 PCS 3 PCS 4 PCS 1 PCS
Art.no 803036 802719 802718 802642 803019 802696 800158 800774 802600 802832 802835 802833 802834 800438 802714 800436 800023 800057	Term Maneuver box Plus f up/down+charger+brake kpl with cabling f Relay card Cabling f Electric kit Brake f Relay card (spiral cable) Cabling f electricity supply f Relaycard L=500mm Cabling f up/down and charging f Relaycard Cable lead-through 11x14x1.5 Maneuver button f Relay card Pressure current adjuster IPC3SAD6 Brake button IP red square Plastic plug Man box Electric chair 50x30 Relay card current limitation Brake 8 amp Relay card box f Relay card Protective cover t Relay card box f Relay card Cover t Relay card box f Relay card Attachment t Relay card box f Relay card K6S M4x6 OBH Socket head cap screw M4x6 oval head screw Bolster plate f Relay card SPAX KXTB 5x20 black-chrome black star squad. H-belt Bundle strap 203x3.6mm black Bundle strap 71x1.8mm black 037 Electric kit f electric brake PLUS-base Term Electric kit REAL f brake f Real 9000 Round pin socket Red 1504 HO Red 0.75-1.65 mm2 Round pin Red 1504 HA 0.75-1.65 mm2 Electric brake Axle attachment Ø20 mm f Electric kit	Amount 1 PCS 2 PCS 4 PCS 3 PCS 4 PCS 3 PCS 4 PCS 1 PCS

2 PCS

1 PCS

1 PCS

Clasp pin 2mm FZB Type B f Cylindrical bolt

Fuse holder Blue f vehicle safety fuse

Fuse 15 amp Blue vehicle safety fuse

801725 802841

802843

ARTICLE LISTING OVER THE KITS FOR CHARGE BOX

NOTE! Complete the appliances below with a charge box.

803029 Charge box Hand Remote Joystick Up/down f relay card Ver 2 803030 Charge box Hand Remote Joystick Multi f relay card Ver 2

802748 Relay card kit up/down+ charger cpl excl. charge box

Art.no	Term	Amount
802681	Cabling f motor 9100 f Relay card	1 PCS
802648	Cabling f battery cpl 9100 f Relay card	1 PCS
803018	Cabling lead-through 10x14x1.5	1 PCS
802546	Relay card current limitation Up-Dpwn/Seat angle 16 amp	1 PCS
802832	Relay card box f Relay card	1 PCS
802835	Protective cover t Relay card box f Relay card	1 PCS
802833	Cover t Relay card box f Relay card	1 PCS
802834	Attachment t Relay card box f Relay card	1 PCS
800438	K6S M4x6 OBH Socket head cap screw M4x6 oval head screw	2 PCS
800023	Bundle strap 203x3.6mm black	4 PCS

802750 Relay card kit backrest angle cpl excl. charge box

Art.no	Term	Amount
802681	Cabling f motor 9100 f Relay card	1 PCS
802718	Cabling f electricity supply f Relay card L=500mm	1 PCS
803019	Cabling lead-through 11x14x1.5	1 PCS
802578	Relay card current limitation Seat/Backrest angle 16 amp	1 PCS
802832	Relay card box f Relay card	1 PCS
802835	Protective cover t Relay card box f Relay card	1 PCS
802833	Cover t Relay card box f Relay card	1 PCS
802834	Attachment t Relay card box f Relay card	1 PCS
800438	K6S M4x6 OBH Socket head cap screw M4x6 oval head screw	2 PCS
802714	Bolster plate f Relay card	1 PCS
802862	MRT-TT FZB M4x6 (screw for charge box)	1 PCS
800023	Bundle strap 203x3.6mm black	4 PCS
800057	Bundle strap 71x1.8mm black	3 PCS
800023	Bundle strap 203x3.6mm black	4 PCS

802731 Electric kit f backrest angle f relay card Real 9000

Art.no	lerm	Amount
802644	Electric kit REAL f backrest angle f Real 9000	1 PCS
801354	Cylindrical bolt fzb	2 PCS
801725	Clasp pin 2mm FZB Type B f Cylindrical bolt	2 PCS
801421	Electric back attachment, adjustable Rear side	1 PCS
801422	Electric back attachment, adjustable Front side	1 PCS
802744	Electric back, back pipe lower f Electric kit	1 PCS
800837	Back tilting attachment Medic Cla/ErgoM	1 PCS
800369	M6S M8x16 8.8 FZB Screw M8x16 FZB	1 PCS
801212	Furniture Nut M6 Rough skull	2 PCS
800507	MPSH M6x38 FZB Pin screw M6x38	1 PCS
801566	MF6S M6x16 OBH 10.9 Socket sink head M6x16	2 PCS
800096	MC6S M6x16 FZB 8.8 Socket head cap screw M6x16 FZB	4 PCS
800462	BRB 6.4x12x1.5 black Plate 6.4x12x1.5 FZB	4 PCS

801265	Back pillar Std. Cla/ErgoM Low back	1 PCS
800841	Back pillar Std. Cla/ErgoM High back	1 PCS

802749 Relay card kit seat angle cpl excl. charge box

Art.nr	Benämning	Antal
802681	Cabling f motor 9100 f Relay card	1 PCS
802718	Cabling f electricity supply f Relay card L=500mm	1 PCS
803019	Cabling lead-through 11x14x1.5	1 PCS
802578	Relay card current limitation Seat/Backrest angle 16 amp	1 PCS
802832	Relay card box f Relay card	1 PCS
802835	Protective cover t Relay card box f Relay card	1 PCS
802833	Cover t Relay card box f Relay card	1 PCS
802834	Attachment t Relay card box f Relay card	1 PCS
800438	K6S M4x6 OBH Socket head cap screw M4x6 oval head screw	2 PCS
802714	Bolster plate f Relay card	1 PCS
802862	MRT-TT FZB M4x6 (screw for charge box)	1 PCS
800023	Bundle strap 203x3.6mm black	4 PCS
800057	Bundle strap 71x1.8mm black	3 PCS

802730 Electric kit f seat angle f relay card Real 9000

Art.no	Term	Amount
802643	Electric REAL f seat angle f Real 9000	1 PCS
801354	Cylindrical bolt fzb	2 PCS
801725	Clasp pin 2mm FZB Type B f Cylindrical bolt	2 PCS

802751 Relay card kit brake cpl excl. charge box

Art.nr	Benämning	Antal
802719	Cabling f Electric kit Brake f Relay card (spiral cable)	1 PCS
802718	Cabling f electricity supply f Relay card L=500mm	1 PCS
803019	Cabling lead-through 11x14x1.5	1 PCS
802600	Relay card current limitation Brake 8 amp	1 PCS
802832	Relay card box f Relay card	1 PCS
802835	Protective cover t Relay card box f Relay card	1 PCS
802833	Cover t Relay card box f Relay card	1 PCS
802834	Attachment t Relay card box f Relay card	1 PCS
800438	K6S M4x6 OBH Socket head cap screw M4x6 oval head screw	2 PCS
802714	Bolster plate f Relay card	1 PCS
800436	SPAX KXTB 5x20 black-chrome black star squad. H-belt	1 PCS
800023	Bundle strap 203x3.6mm black	4 PCS
800057	Bundle strap 71x1.8mm black	3 PCS

803037 Electric kit f electric brake PLUS X/H-base Ø20 Axle

Art.no	Term	Amount
802645	Electric kit REAL f brake f Real 9000	1 PCS
802706	Electric brake Axle attachment Ø12mm f Electric kit	1 PCS
802707	Electric brake Centre attachment 9000 f Electric kit	1 PCS
801354	Cylindrical bolt fzb	2 PCS
801725	Clasp pin 2mm FZB Type B f Cylindrical bolt	2 PCS

ARTICLE LISTING OVER APPLIANCES FOR CHARGE BOX

















	DETAILS
CHARGEBOX	DEIAILS

1	Hand control Up/Down	802739
2	Hand control Multi	802740
3	Remote control, Transmitter Up/Down	801806
4	Remote control, Transmitter Multi	801808
5	Remote control, Kit Up/Down	802686
6	Remote control, Reciever Up/Down	802737
7	Remote control, Reciever Multi	802738
8	Remote control, Kit Multi	802687
9	Foot control electric lever	802688
10	Joystick 3 func. + brake	802743
	Joystick 2 func. + brake	802742
	Joystick 2 func. no brake	802741







Mercado Medic AB Box 1074 181 22 Lidingö

Phone & Fax Ph +46 8 555 143 00 Fax + 46 8 555 143 99

Art.No

Email & Internet
Email info@mercado.se
Internet www.mercado.se